

The ContacEZ **IPR Strip System** is a safe and innovative interproximal reduction system that makes interproximal enamel stripping efficient and accurate without creating sharp corners or subgingival ledges. Designed for use with clear aligners and other orthodontic treatments, these flexible diamond strips curve and conform to the natural contours of the teeth to maximize patient comfort and safety.

- ▶ Ergonomic, single-handed design
- ▶ Provides optimal tactile control
- ▶ Will not remove excess enamel
- ▶ Will not cut lips or gums
- ▶ Prevents sharp corners or subgingival ledges
- ▶ Effective for anterior and posterior areas
- ▶ Central opening for maximum visual perception
- ▶ Saves chair-time

IPR Strip System



Yellow Starter • 0.07 mm
Single-Sided • Extra-Fine Diamond • Serrated

REF 32108



Red Opener • 0.12 mm
Double-Sided • Medium Diamond

REF 32408



Dark Blue Widener • 0.15 mm
Double-Sided • Coarse Diamond

REF 32508



Green Extra-Widener • 0.20 mm
Double-Sided • Coarse Diamond

REF 32608

IPR Plus Strips



0.10 mm Single-Sided Opener
Clear • Med-Fine Diamond • Serrated

REF 32208



0.12 mm Single-Sided Opener
Cyan • Medium-Coarse Diamond

REF 32308




0.25 mm Double-Sided Widener
Purple • Coarse Diamond

REF 32708



0.30 mm Double-Sided Widener
Brown • Coarse Diamond

REF 32808

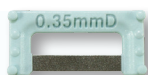


0.35 mm Double-Sided Widener
Mint • Coarse Diamond

REF 36608



0.40 mm Double-Sided Widener
Aqua • Coarse Diamond

REF 36708

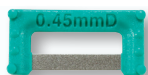

0.45 mm Double-Sided Widener
Turquoise • Coarse Diamond

REF 36808



0.50 mm Double-Sided Widener
Teal • Coarse Diamond

REF 36908

Usage Guidelines & Technique

This technique is especially effective for anterior teeth and for case refinement. ContacEZ **IPR Strips** are autoclavable (**steam heat only, no dry heat, up to 270°F for 10 minutes at 30 psi max**) and reusable, and should be sterilized prior to the first use. The manufacturer recommends to use each strip up to 5 times for maximum efficiency and functionality.

- ① Hold the strip with your thumb and forefinger, insert it into the interproximal space, and gently pass back and forth buccolingually until the strip is passive. Do not use vertical force. Start with the thinnest diamond strip, and repeat with thicker strips until you have reduced (stripped) the correct amount of enamel according to the treatment plan.
- ② To ensure accurate measurements, polish the proximal surface using the ContacEZ **Gray Final Polishing Strip**.
- ③ Verify reduction achieved using the ContacEZ **Incremental Thickness Gauge**.
- ④ **Pediatric dentists** can use ContacEZ **IPR Strips** for the serial proximal reduction of primary molars.

For convenience when autoclaving and organizing, store strips in a ContacEZ Strip Tray



Patented
contacez.com

100% Guaranteed
360.694.1000



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands



ContacEZ, LLC
217 SE 136th Ave, Suite 105
Vancouver, WA 98684
United States