

INSTRUCTIONS FOR USE



Molar Component

Position molar component and contour the bar to the arch shape. Bend molar end of bar if additional molar rotation is desired.



Mark the bar to the center of the cuspid (or 1st bicuspid)

Cut off excess end of bar and verify bar fits fully into anterior component hole. Etch teeth to be bonded and clean molar pad of saliva contamination. You can add relief wax to the posterior component so it does not move during bonding.

Bond molar component. Slide anterior component onto bar and bond to tooth. Repeat bonding procedure for opposite side. Remove relief wax.

Band or direct bond mandibular molar with buccal tube or button. Instruct the patient to wear 1/4" 6.5 ounce elastics for 24 hour wear and evaluate position in 6-8 weeks. The D2 Appliance is usually worn 4-6 months and adjust elastic force as desired.

Anterior Component

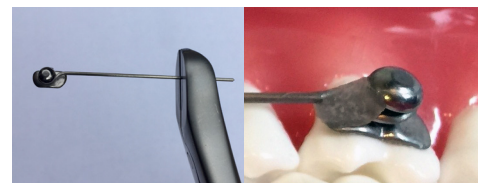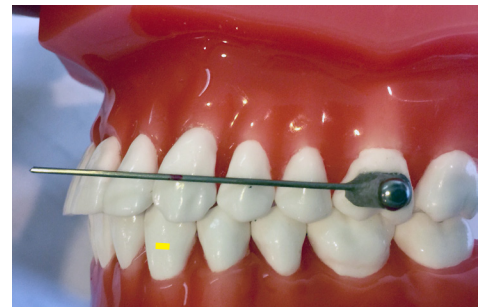

A lower ESSIX retainer or lingual arch is recommended with the D2 Appliance. Cut out a window on the lower ESSIX retainer where the buccal tube or button will be placed.